



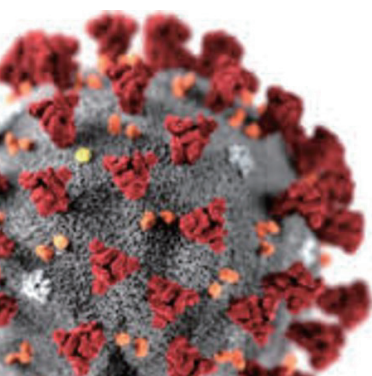


# ichroma™ COVID-19 Ab test

## Anti-viral IgM/IgG test

- Fast and precise result (10 mins)
- Small POC device
- Room temperature storage
- Whole blood, Serum and Plasma (10 µL)

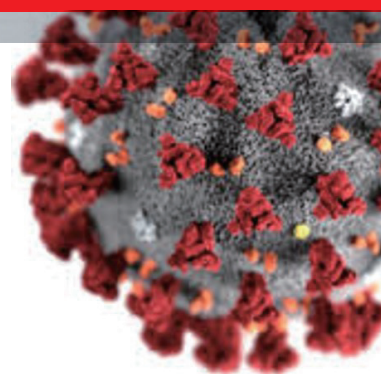

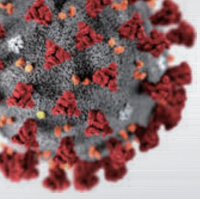

# ichroma™ COVID-19 Ab test


Add sample into  
detector tube

Shake 10 times

Draw & load sample mixture

Wait 10 minutes



Read the test result

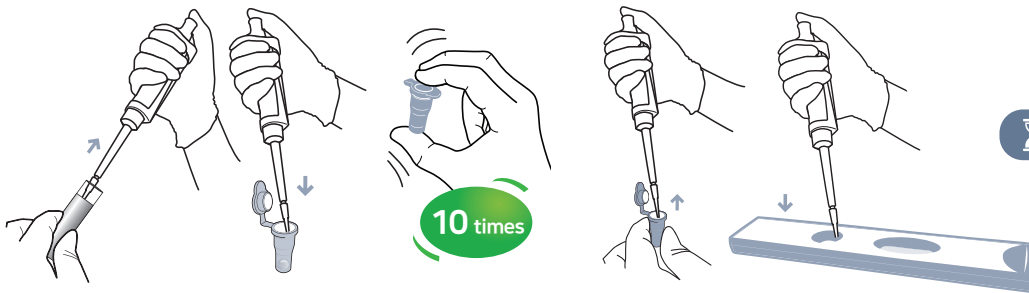

10 Min



## • High sensitivity (Time resolved fluorescence) immunoassay

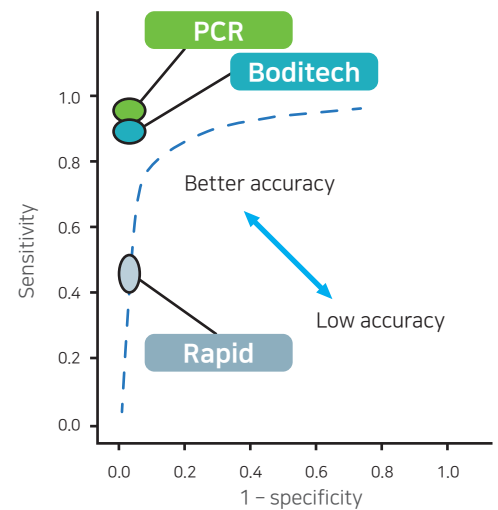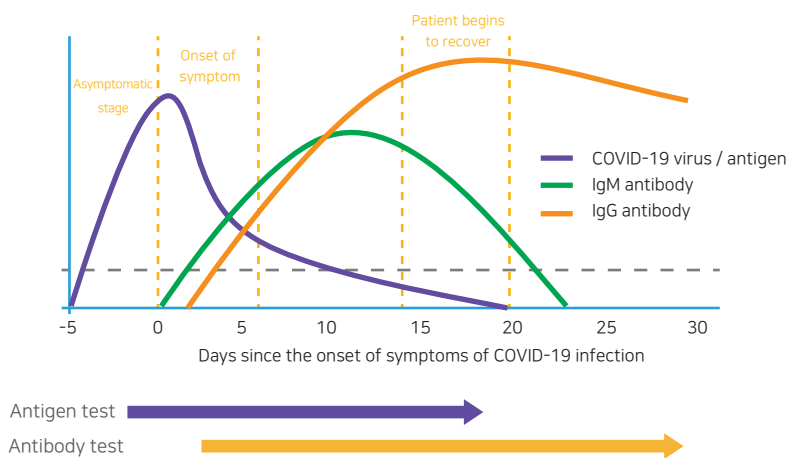

\* Based on the clinical data with ichroma™ II Influenza A+B reagent

## • Clinical performance

		RT-PCR		
		Positive	Negative	Total
ichroma™ COVID-19 Ab	Positive	46	0	46
	Negative	0	131	131
	Intermediated	2	4	6
	Total	48	135	183
		Clinical sensitivity: 95.8%		
		Clinical specificity: 97.0%		

Product name	ichroma™ COVID-19 Ab
Assay type	TRFLFA*, IgM/IgG
Sample type	Whole blood/Serum/Plasma
Sample prep / extraction	Fingertip blood / tube blood
Target	Anti-viral IgM/IgG
Platform	ichroma™ II Reader
Unit	25 kit/box

\* TRFLFA: Time-resolved fluorescent lateral flow assay

## • Ordering Information

Product	Cat. No.
ichroma™ COVID-19 Ab	CFPC-114
ichroma™ II Reader	FPRR021

