


Boditech Quick COVID-19 Ag

No pain Saliva COVID-19 test

Easy saliva based test

- No painful swab
- Simple 1 step: collection and drop
- Fast and accurate result

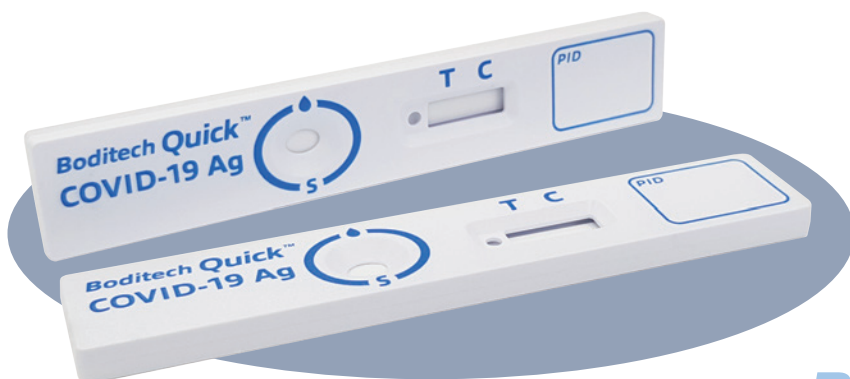

Boditech Quick™


Find out more on www.boditech.co.kr


• COVID-19 Saliva test - Boditech Quick COVID-19 Ag

Most COVID-19 assays require nasopharyngeal swab samples that could be painful and require skills. Recently, it is reported that saliva and nasopharyngeal SARS-CoV-2 results exhibit correlation. Boditech Quick COVID-19 Ag is saliva based COVID-19 test with specially designed saliva collector for an accurate test. With Boditech Quick COVID-19 Ag saliva collector, saliva sampling is easy and painless.


• Test procedure

1 Place the sponge in your mouth and chew it gently for about 30 seconds until it softens. Then, wait 2 minutes for the sponge to get wet with saliva.

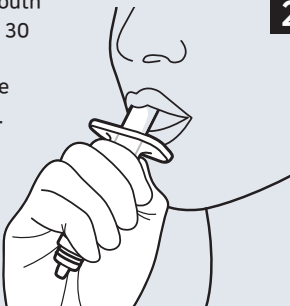


 **2-3 Min**

2 When sufficient saliva is collected and the sponge softens, put the wet sponge into the body of the barrel.

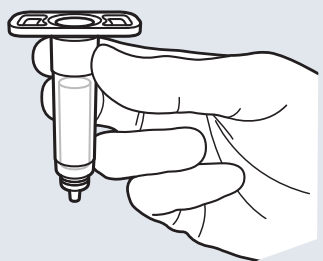


3 One drop at a time

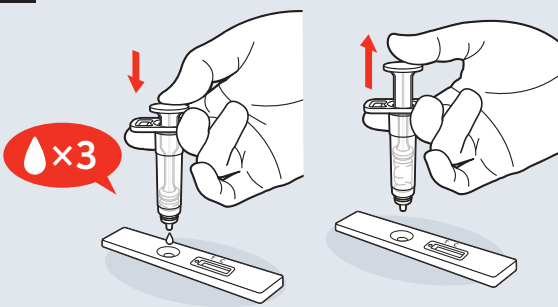


4 Read the result in 10 - 25 minutes.


Negative




Positive



Invalid



Invalid



Strong positive results are confirmed in 10 minutes.

• Specification

Product	Boditech Quick COVID-19 Ag
Test principle	Gold rapid test
Test type	Qualitative (positive/negative)
Sample type	Saliva
Test time	15 - 30 min (including sampling time)
Storage	4-30℃
Performance	Sensitivity 80% Specificity 100%
Kit component	Saliva collector, sponge, cartridge